







## ResMed AirTouch N20/N20 for Her Nasal Mask

### Cleaning/Replacement Guidelines

	ResMed Product#	HCPCS Code	Replacement Schedule
 <b>AirTouch N20</b> or <b>AirTouch N20 for Her mask</b> (with headgear)	AirTouch N20 mask, standard headgear with cushion size: <ul style="list-style-type: none"> <li>• <b>S - 63903</b></li> <li>• <b>M - 63901</b></li> <li>• <b>L - 63902</b></li> </ul> AirTouch N20 for Her mask, small headgear with cushion size: <ul style="list-style-type: none"> <li>• <b>S - 63900</b></li> </ul>	<b>A7034 + A7035</b> (Mask + Headgear)	
 <b>AirTouch N20 Frame System</b>	AirTouch N20 frame with cushion size: <ul style="list-style-type: none"> <li>• <b>S - 63953</b></li> <li>• <b>M - 63955</b></li> <li>• <b>L - 63956</b></li> </ul> AirTouch N20 for Her frame with cushion size: <ul style="list-style-type: none"> <li>• <b>S - 63954</b></li> </ul>	<b>A7034</b> (Mask)	Replace every <b><u>3 months.</u></b>
 <b>AirTouch N20 Headgear</b>	AirTouch N20 Headgear <ul style="list-style-type: none"> <li>• <b>Small - 63560</b></li> <li>• <b>Standard - 63561</b></li> <li>• <b>Large - 63562</b></li> </ul> AirTouch N20 for Her Headgear <ul style="list-style-type: none"> <li>• <b>Small - 63558</b></li> </ul>	<b>A7035</b> (Headgear)	Replace every <b><u>6 months.</u></b>
 <b>AirTouch N20 Cushion</b>	AirTouch N20 cushion size: <ul style="list-style-type: none"> <li>• <b>S - 63950</b></li> <li>• <b>M - 63951</b></li> <li>• <b>L - 63952</b></li> </ul>	<b>A7033</b> (Cushions)	Replace every <b><u>month.</u></b>  May replace every 2 weeks if needed.
<b>Cleaning Schedule</b>	<b>DO NOT WASH AIRTOUCH N20 CUSHION</b> Wash frame, short tube, and headgear <u>weekly</u> with warm water and mild soap and air dry. Wipe cushion with moist wipes weekly or sooner if needed.		

**DME Provider:**  
**Phone Number:**