









## ResMed AirSense 11

### Cleaning & Replacement Guidelines

	ResMed Product #	HCPCS Code	Cleaning Schedule	Replacement Schedule
 <b>AirSense 11</b>		E0601 + E0562	Wipe weekly the exterior surface with a dry cloth.	Every <b>5 years</b> .
 <b>Standard Air Filter</b> or  <b>Hypoallergenic Air Filter*</b>	39301 (2-pack) 39305 (2-pack)	A7038 A7038	Check weekly for dirt or blockage.  Do NOT wash.	Replace at least every <b>6 months</b> .  May be replaced every 2 weeks if needed.
 <b>SlimLine Tubing</b> or  <b>ClimateLineAir 11 Heated Tube</b>	36810 39102	A7037 A4604	Clean weekly with warm water and mild soap and air dry.	Replace every <b>3 months</b> .
 <b>HumidAir 11 Water Tub</b>	39100	A7046	Clean weekly with warm water and mild soap and air dry.	Replace every <b>6 months</b> .

Standard air filter efficiency > 75% for ~7 micron dust.

\*Hypoallergenic air filter > 98% for ~7-8 micron, > 80% for 0.5 micron dust

DME provider:  
Phone number: