


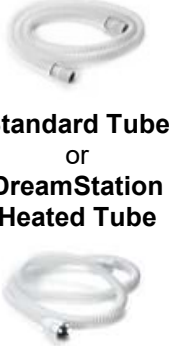





Philips DreamStation

Cleaning & Replacement Guidelines

	Philips Product #	HCPCS Code	Cleaning Schedule	Replacement Schedule
 DreamStation		E0601 + E0562	Wipe weekly the exterior surface with a dry cloth.	Every 5 years .
 DreamStation Pollen Filter	1122446	A7039	Rinse weekly with water and air dry.	Replace every 6 months .
 DreamStation Ultra-Fine Filter (optional)*	1122518 (2-pack)	A7038	Check weekly. Do NOT wash.	Replace every month or sooner when damaged or dirty.
 Standard Tube or DreamStation Heated Tube	PR15 HT15	A7037 A4604	Wash weekly with warm water and mild soap and air dry.	Replace every 3 months .
 DreamStation Water Tank	1122520	A7046	Wash weekly with warm water and mild soap and air dry.	Replace every 6 months .

*Pollen filter efficiency > 88% for 7-10 micron dust.
Ultra-fine filter > 95% for 0.5-0.7 micron dust.

**DME provider:
Phone number:**